stübben
— x —
EQUI
c o m p r e s s

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Informationsbroschüre

*Compression stockings
for horses*

Explanatory booklet

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Summary

<i>Introduction</i>	<i>1</i>
<i>Set content.....</i>	<i>2</i>
<i>How does the set work?.....</i>	<i>7</i>
<i>Areas of application.....</i>	<i>10</i>
<i>Contraindication.....</i>	<i>11</i>
<i>How to put it on.....</i>	<i>12</i>
<i>Size chart</i>	<i>14</i>
<i>How to care for EQUI compress</i>	<i>16</i>
<i>Impressum</i>	<i>17</i>

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Introduction

Thank you for buying the Stübben x EQUI compress horse compression stockings!

This booklet will give you the keys to properly use them. Please read carefully for best results..

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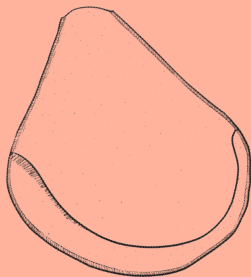
Set content



In the Stübben x EQUI compress set, you will find the following elements:

- *compression stockings for the horse, going from right under the knee down to the hoof*
- *compression boots, either for the front or hind legs, to wear on top of the stockings*
- *a dressing aid to help put the stockings on*
- *a wash and storage cotton bag*

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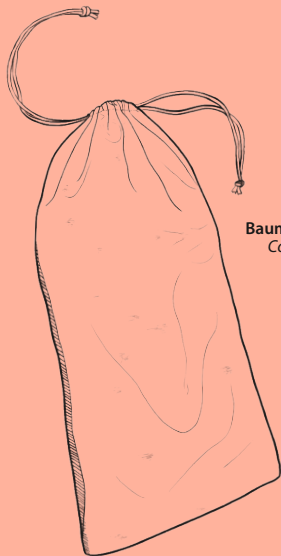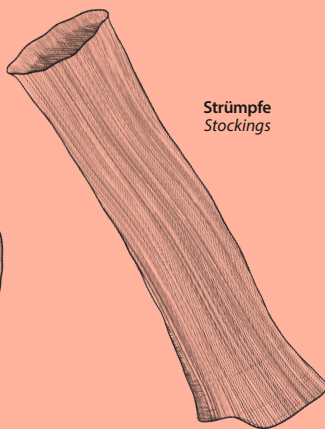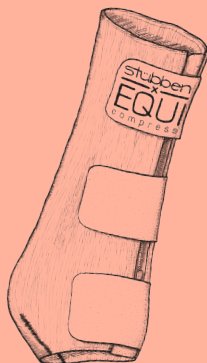
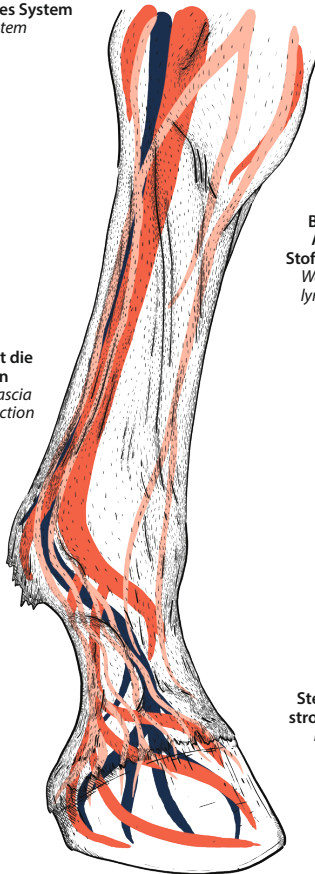
**Kompression vom
Kronrand beginnend**
*Compression starting
from the coronet*

How does the set work?

Stübben x EQUI compress results from the scientific work of equine physiotherapist Franzi Bechberger-Gaber, in collaboration with veterinarians and experts in the field of lymphology.

When a horse is immobile for long periods or is trained at an intensive rate, the lymphatic system and its metabolic waste evacuation might not function correctly, leading to leg swelling and slower tissue regeneration.

The compression stockings Stübben x EQUI compress were designed to counter this effect. They stimulate the lymphatic system through optimal pressure and effective pumping function. With the help of elaborate test procedures, the optimal pressure values were determined to positively influence the horse's vascular system and enable preventive measures to be taken.

Compression therapy involves applying pressure from the extremities towards the heart to narrow blood vessels. This enhances blood circulation, improving the flow back to the heart and preventing oedema or swelling. It also helps restore the functionality of the valves in veins and lymphatic vessels by bringing them closer, thus speeding up the removal of metabolic waste from the body.

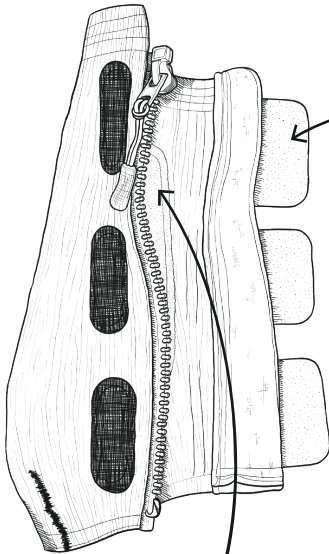
Furthermore, EQUI compress stimulates the fascia, improving tissue health, and relieves tension in the tendons with precise pressure application. Overall, it reduces congestion in the extremities, providing relief and well-being.

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EQUI compress is knitted from silver ions infused yarn that prevents bacteria buildup. This helps protect irritated pastern tissue. The breathable material effectively manages moisture while providing a cooling sensation.


Inside of the boot, EQUI pelotte applies pressure to facilitate the removal of metabolic wastes from the extremities without compressing the vessels. Its texture, made of rounded bumps, stimulates the lymphatic system and fascia. Plus, when the horse moves, the resulting pump effect is conveyed to the suspensory ligament by EQUI pelotte.

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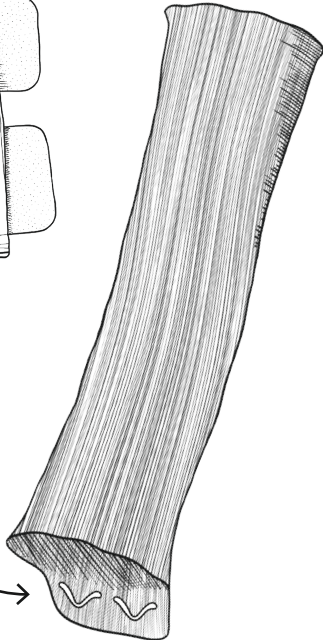Areas of application

EQUI compress is perfect for aiding recovery in resting or immobilised horses. It is ideal:

- *for horses in resting phases or for stabled horses*
- *during transport and at horse shows*
- *during intense training phases*
- *against swollen legs and windgalls*

EQUI compress has been designed for rest phases rather than training, as movement and exercise already sufficiently stimulates the circulatory system.

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Contraindication

Do not use EQUI compress with:

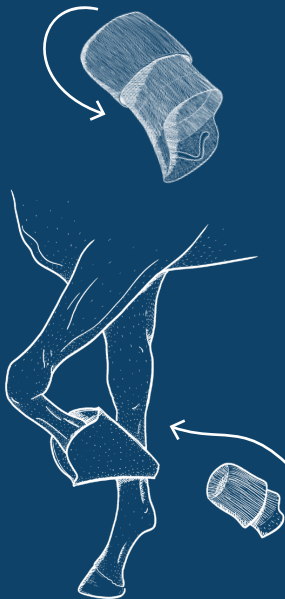
- *open wounds*
- *bacterial infections*
- *fungal infections*
- *blood flow disorder*
- *severe inflammation*
- *fever*
- *(venous disease and raised temperature)*

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How to put it on

Follow the steps carefully to make the process easier!

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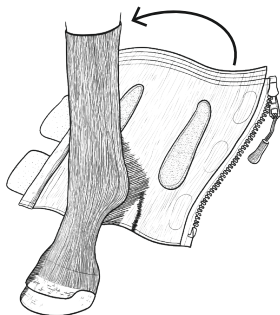
Place the boot over the stocking and tightly fasten the middle strip

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Lay the zipper flap flat, then zip down to the middle strip

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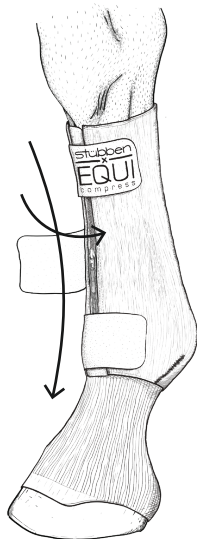
Fasten the top and bottom strips and open the middle one

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Zip the boot all the way and fasten the middle strip

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Turn the top part of the stocking over the boot

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Size chart

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Size chart front


	S	M	L	XL
Röhrbein unter Gelenk (A) <i>Cannon under the joint (A)</i>	< 18,5 cm	18,5 - 20,5 cm	20,5 - 23 cm	> 23 cm
Röhrbein Mitte (B) <i>Middle of the cannon (B)</i>	< 18 cm	18 - 20 cm	20 - 22 cm	> 22 cm
Fesselgelenk (C) <i>Fetlock (C)</i>	< 26 cm	26 - 28 cm	28 - 30 cm	> 30 cm
Pferdegröße <i>Horse size</i>	Pony	Vollblut <i>Cob</i>	Warmblut <i>Full</i>	Extra-Groß <i>Extra full</i>

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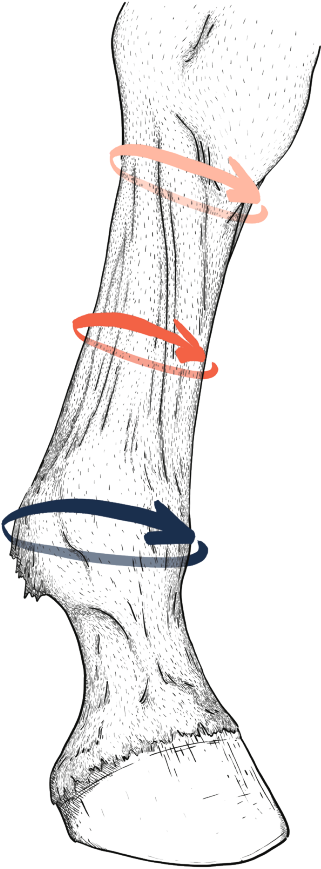How to care for EQUI compress

Only use the boots and the stockings when they are dry and clean so that the silicone and antibacterial material remain functional. The horse's legs must be cleaned and dried before putting them on. When the horse needs EQUI compress for an extended period, please check daily that no pressure marks have occurred (for example because of a piece of straw that got caught under the stockings).

EQUI compress is machine washable at 30° Celsius. Do not tumble dry. Do not iron.

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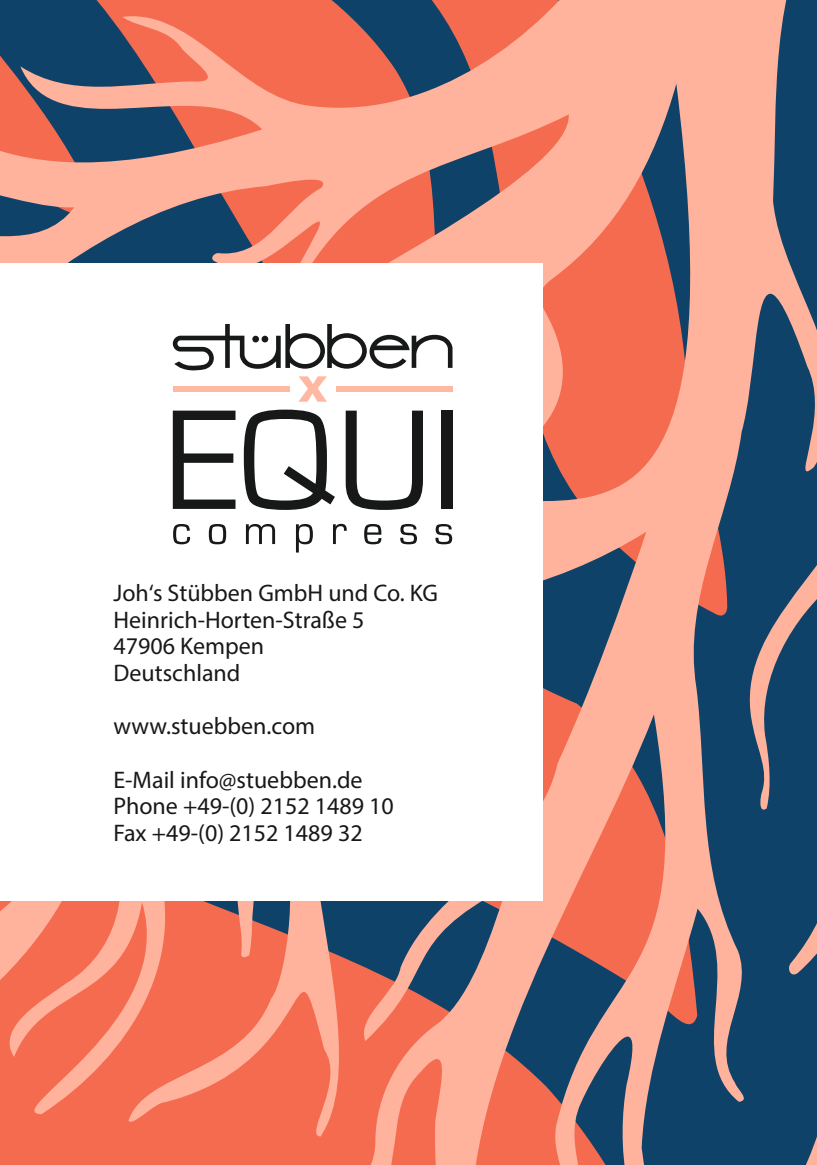
*Printed by FLYERALARM GmbH
Alfred-Nobel-Str. 18
97080 Würzburg
Germany*

Design and photo credits: Joh's Stübben GmbH & Co. KG

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Design und Bildnachweise: Joh's Stübben GmbH & Co. KG
*Address: Joh's Stübben GmbH & Co. KG
GmbH, Amtsgericht Krefeld, HRB 6997*
Joh's Stübben GmbH und Co. KG