WALDHAUSEN

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- DE** Gebrauchsanweisung Waldhausen Health + Care TecSupreme Heiz- und Massagedecke
- GB** Instructions for use for Waldhausen Health + Care TecSupreme Heating and Massage Rug
- FR** Mode d'emploi des Waldhausen Health + Care TecSupreme Couverture chauffante et massante
- IT** Istruzioni per l'uso Waldhausen Health + Care TecSupreme Coperta riscaldante e massaggiante
- ES** Instrucciones de uso Waldhausen Health + Care TecSupreme Manta calefactora y de masaje
- NL** Gebruiksaanwijzing Waldhausen Health + Care TecSupreme Verwarmings- en massage deken
- DK** Brugsanvisning Waldhausen Health + Care TecSupreme Varme- og massage tæppe
- FI** Käyttöohje Waldhausen Health + Care TecSupreme Lämmitys- ja hierontapeitto
- SE** Bruksanvisning Waldhausen Health + Care TecSupreme Värme- och massagefilt
- NO** Bruksanvisning Waldhausen Health + Care TecSupreme Varme- og massasjedyne
- PL** Instrukcja obsługi Waldhausen Health + Care TecSupreme Koc grzewczy i masujący
- BG** Инструкция за употреба Waldhausen Health + Care TecSupreme Загриващо и масажирско одеяло
- CZ** Instrukce k použití Waldhausen Health + Care TecSupreme Vyhřívací a masážní deka
- EE** Kasutusjuhend Waldhausen Health + Care TecSupreme Soojendus- ja massaažitekk
- GR** Οδηγίες Χρήσης Συσκευής Μασάζ Waldhausen Health + Care TecSupreme Θερμαινόμενη και μασάζ κουβέρτα
- HR** Upute za uporabu Waldhausen Health + Care TecSupreme Grijači i masažni pokrivač
- HU** Használati útmutató Waldhausen Health + Care TecSupreme Fűtő- és masszázstakaró
- LT** Gamybos instrukcija Waldhausen Health + Care TecSupreme Šildymo ir masažo antklodė
- LV** Lietošanas instrukcija Waldhausen Health + Care TecSupreme Sildīšanas un masāžas sega
- MT** Istruzzjonijiet ta' Użu ġenerali għal Waldhausen Health + Care TecSupreme Ħarqa u blankets ta 'massage
- PT** Manual de Instruções Waldhausen Health + Care TecSupreme Cobertor de aquecimento e massagem
- RU** Инструкция по применению Waldhausen Health + Care TecSupreme Обогревающее и массажное одеяло
- SI** Navodila za uporabo Waldhausen Health + Care TecSupreme Ogrevalna in masažna pregrinjala
- SK** Návod na použitie Waldhausen Health + Care TecSupreme Vyhrievacia a masážna deka

Art.-Nr./Product no./N° Art.:
389001

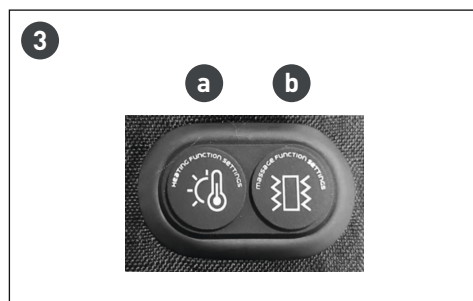
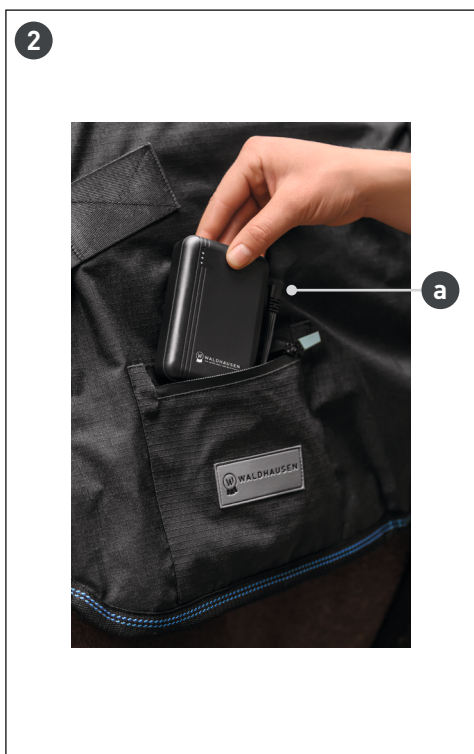
WALDHAUSEN
**HEALTH
+ CARE**

www.waldhausen.com

Waldhausen GmbH & Co. KG
Von-Hünefeld-Str. 53 | 50829 Köln | Germany
info@waldhausen.com

Inverkehrbringer:

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Scope of delivery

1 x heated massage blanket
 instructions for use
 storage bag

Technical details

Model: Waldhausen Health + Care TecSupreme heated and massage blanket		
Dimensions: Size M – up to approx. 140 cm Size L – up to approx. 155 cm		
Cover material: Outer material 100% polyester (600D ripstop), inner material 100% polyester		
7 massage heads on each side, each adjustable in 3 stages		
1 heating system with 7 heating zones on each side, each adjustable in 3 stages		
2 slip pockets (left/right) with zip fastener, including cable with barrel connector for connecting power banks		
2 elastic ceiling straps		
1 removable chest strap with removable fur cover (100% polyester)		
1 removable hind leg strap		
Temperature protection via NTC sensor, continuous measurement and automatic temperature adjustment		
3 vibration levels Red: non-stop White: short interval Blue: long interval	3 heating levels Red: 65 °C White: 55 °C Blue: 45 °C	Maximum heating/massage duration: 2-8 hours, depending on the heating and vibration intensity
Heating duration with 7.4 V 5000 mAh power bank (approx.)		
Vibration only: Red: 7 hours White: 7 hours 30 minutes Blue: 8 hours	Heating only: Red: 2 hours 40 minutes White: 3 hours 10 minutes Blue: 3 hours 40 minutes	Heating + vibration simultaneously: Red: 2 hours White: 2 hours 30 minutes Blue: 3 hours
The power bank is not included in the scope of delivery. Please order item 2 x 389100.		

1. General instructions for use

- Please read the instructions for use before using for the first time.
- Please check that the delivery is complete and undamaged. Only use and clean the device when it is undamaged and dry, and in a dry environment.
- Improper operation and use of this device may result in injury and damage. By using this device, you are deemed to have understood, acknowledged and accepted all the terms and conditions contained in this document.
- For private use only, not for medical or commercial purposes.
- The device is only intended for users aged 8 years and above. Users with physical, sensory or mental impairments may only operate the device under supervision.
- The device is intended exclusively for use on horses in the area of large muscle groups.
- The use of the device should be perceived as pleasant for the animal at all times. In all other cases, please discontinue use immediately.
- In case of illness or injury, e.g. inflammation, allergies, coat irritation, after surgery, during pregnancy: consult your vet and/or physiotherapist before use.
- Only use the device when the horse is awake.
- The manufacturer shall not be liable for damage caused by improper use.

2. Applications

Well-being at the touch of a button. The Waldhausen Health + Care TecSupreme heating and massage blanket specifically relaxes tense muscles, promotes blood circulation and supports regeneration. Each side of the blanket has its own heating system and seven integrated massage buttons. Both sides can be controlled individually and independently of each other. Quick, effective and super easy to use. Whether before training, after work or simply to pamper your horse, it will experience warmth and massage as if it were in its own wellness spa. Battery life: three to eight hours, depending on the heating and vibration intensity.

- **Muscle relaxation:** Heat promotes blood circulation and helps to relax tense muscles – especially before training or after strenuous work.
- **Improved regeneration:** The massage function supports lymph flow and can accelerate the removal of metabolic waste products.
- **Pain relief:** Heat can relieve tension and cold-related pain, for example in older horses or animals with muscular problems.
- **Stress relief:** Gentle massage and even heat have a calming effect and can increase well-being.
- **Prevention:** Regular use can help prevent muscle stiffness and injuries.
- **Before training:** Apply shortly before riding or working to gently warm up the muscles and reduce the risk of injury.
- **After work:** Use after exertion to promote regeneration and relaxation.
- **In cold weather:** To keep muscles warm during the transition period or in winter, e.g. for clipped horses.
- **Rehabilitation:** As a supportive measure in consultation with a vet or physiotherapist for muscle tension, back problems or after injuries.
- **Wellness & care:** Even without any specific reason, as a pampering programme for the horse to promote well-being and bonding.

3. Application / Commissioning

Fig. 1. Ceiling

- Message points
- Heating points
- Zippered pockets
- Control panel

Fig. 3. Control panel

- Display/operation of heating panel
- Display/Operation Massage

Fig. 2. Zippered insert pocket for battery

- Hollow plug connection

Fig. 4. Storage bag

Commissioning:

1) Fully charge both Waldhausen Powerbanks 389100; charging time approx. 2 hours. The Waldhausen Powerbank 389100 has an on/off button for the charge level indicator. Please use the enclosed USB-C cable with a suitable adapter. The indicators flash during charging. When the device is fully charged, all indicators light up. Please disconnect the fully charged device from the charging unit immediately.

2) Spread out the blanket, open the zip pockets, connect the power bank to the blanket using the connection cable, and close the zip pocket. Switch on the heat/vibration function on the control panel. Please select the button for the desired function. To do this, press and hold the button for 3-5 seconds. The blanket will then switch on at the first level. The intensity levels can be conveniently changed by pressing the button repeatedly.

3 vibration levels

	Red: non-stop
	White: short interval
	Blue: long interval

3 heat settings

	Red: 65 °C
	White: 55 °C
	Blue: 45 °C

To switch off, press the buttons on the control panel for 3 seconds.

3) Please familiarise yourself with the product and its functions before using the blanket on the horse. Ideally, you should clean the horse before use. When using the blanket on the horse for the first time, it is recommended that a second person hold the horse in place rather than tying it up. Place the blanket on the horse, do not fasten the straps, start with the heat function at the lowest setting, and once the horse has become accustomed to it, add the massage at the lowest setting. Pay attention to the horse's reaction; if in doubt, remove the blanket immediately. Gradual acclimatisation. After an acclimatisation phase, you can fasten the straps.

4. Safety instructions

- Before use, please check that the blanket and power bank connections are intact.
- Do not use the blanket while charging.
- The ceiling must not be altered, repaired or opened. If you encounter any problems, please contact your dealer or the manufacturer.
- Keep/use the blanket and power bank 389100 out of the reach of children and vulnerable groups.
- Protect the blanket and power bank from extreme temperatures, direct sunlight, fire, heat, magnets and moisture. Do not use the device under pillows or covers.
- Ensure that the moving parts are free and that no hair is caught in them.
- Avoid contact with pointed and sharp objects, acids and flammable liquids.
- Do not use any other attachments or cables.
- Fully charge power banks to ensure optimal performance.
- Caution! If the device emits an unusual odour, becomes hot or exhibits other abnormalities, discontinue use immediately.
- Do not leave the horse unattended during application.

5. Storage, cleaning, disposal

- After use, please disconnect the power bank from the ceiling and store it in a dry place.
- After use, fold the blanket and store it in the bag in a dry place.
- Clean the cover with a damp cloth. Please do not wash, dry clean, tumble dry or iron.
- Please observe the care instructions on the label.

6. Warranty

The manufacturer provides a 2-year warranty from the date of invoice for countries within the European Union. Statutory warranty rights remain unaffected. In the event of a warranty claim, please contact your dealer with proof of purchase.

Exclusion of warranty claims: Damage caused by normal wear and tear, environmental influences, improper use or failure to observe safety instructions.

Declaration of conformity: Waldhausen GmbH & Co. KG hereby declares that the product complies with the relevant EU directives and is CE certified.

7. Troubleshooting

Excessive temperature	Switch off the device immediately and remove it from the animal. Cool the blanket at normal room temperature.
Battery power is '0'	Recharge the battery.
The ceiling is making unusual noises.	Switch off the device, check the connections, press and hold the On button.
Device does not start	Check the battery status and recharge if necessary.
Device cannot be charged	Check the USB-C port and power source

8. Disposal

Please do not dispose of the product in household waste. Dispose of it at collection points for electrical appliances.

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Waldhausen GmbH & Co. KG,
Von-Hünefeld-Str. 53, 50829 Cologne, Allemagne
+49 221-58801-0
info@waldhausen.com
www.waldhausen.com

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WALDHAUSEN GmbH & Co. KG